

Relieve with **Comfort Tape!**

comforttape™

orthocare essentials™
Tape for Teeth & Gums

healthy mouth
all natural

sea salt minerals &
functional elements



instant aid for
teeth & gums
up to 1 hour
or more

6 strips

Alleviates & Eases Irritations

Comforts Achy Teeth & Sore Gums

Great for braces, aligners & more!



Relieve with **Comfort Tape!**

comforttape™

orthocare essentials™

Comfort Tape innovative formulation was created by an orthodontist to provide a science and research driven solution for discomfort while wearing braces and aligners. Our formulation is one of the first ever to provide ingredients that alleviates irritation of inflamed gingiva and eases sensitive enamel.



- Made with natural & functional ingredients
- Delivers the best formulation up to 1 hour or more
- Helps comfort and ease teeth & gums
- Safe to use under mouth guards

Scan to Order



Here's how to use Comfort Tape

1. Open comfort tape and peel off one piece holding it by edges.
2. Dry area with tissue or gauze before applying for best adhesion.
3. Store remaining Comfort Tape in pack and seal.
4. Place Comfort Tape on teeth, using perforated lines to center on the problem area.
5. Press over braces or aligners, pushing above and below onto gums and teeth.
6. Keep tape on for 1 hour or more.
7. Don't eat or drink at this time to maximize effects of comfort.
8. Peel off anytime and eat and drink normally.

All Rights Reserved Patent Pending Made in USA Manufactured for: ORTHONU, LLC. Rumson, NJ 07760

