

Comfort & Soothe with **mouth-aid!**



Comfort & Soothe with **mouth-aid!**

mouth-aid™

orthocare essentials™

Aid for Soft Tissue in Oral Cavity

healthy mouth
natural



sea salt minerals &
functional elements



instant relief
for sores

for up to 1-2 hours
or more

6 mouth-aids

Protects Soft Tissue of Inner Lips & Cheeks

Soothes Ulcers & Irritations
Great for braces, aligners & more!



mouth-aid™

orthocare essentials™

mouth-aid innovative formulation was created by an orthodontist to provide a science and research driven solution for discomfort while wearing aligners and braces. **mouth-aid** is one of the first-ever functional ingredient soothing application for any irritation in your mouth. Whether it's for braces or aligners during orthodontic treatment, you can use **mouth-aid** as many times as needed.



- Made with functional & natural ingredients
- Delivers the best formulation up to 1 hour or more
- Soothe & protect mouth, inner lips & cheeks
- Protects soft tissue in oral cavity

Scan to Order



Heres how to use mouth-aid

1. Open a pack of mouth-aid and dry irritated area.
2. Peel off 1 mouth-aid holding it by the edges.
3. Store remaining mouth-aid in pack and seal.
4. Place mouth-aid on localized spot that feels uncomfortable or irritated and press on area
5. Keep strip on for 1-2 hours or more.
6. Don't eat or drink at this time to maximize effects of soothing.
7. Peel off at any time and eat and drink normally.



All Rights Reserved Patent Pending Made in USA Manufactured for: ORTHONU, LLC. Rumson, NJ 07760

Lot 030424 Use by 2026-03



8 50035 48903 1